


Ingro Mate™
PRECISION INGROWN
NAIL INSTRUMENT

**PRECISION
INGROWN NAIL
INSTRUMENT**

FOR THE PRACTITIONER


INGRO MATE
IS THE VERY
LATEST
IN A RANGE
OF EXACTING
ST/STEEL
INSTRUMENTS
DESIGNED FOR
THE MOST
COST EFFECTIVE
TREATMENT
AVAILABLE YET
FOR
THE COMMON
INGROWN NAIL
CONDITION.

- DURABLE FOR REPEATED TREATMENTS
- SUITABLE FOR ALL STERILIZATION PROCESSES

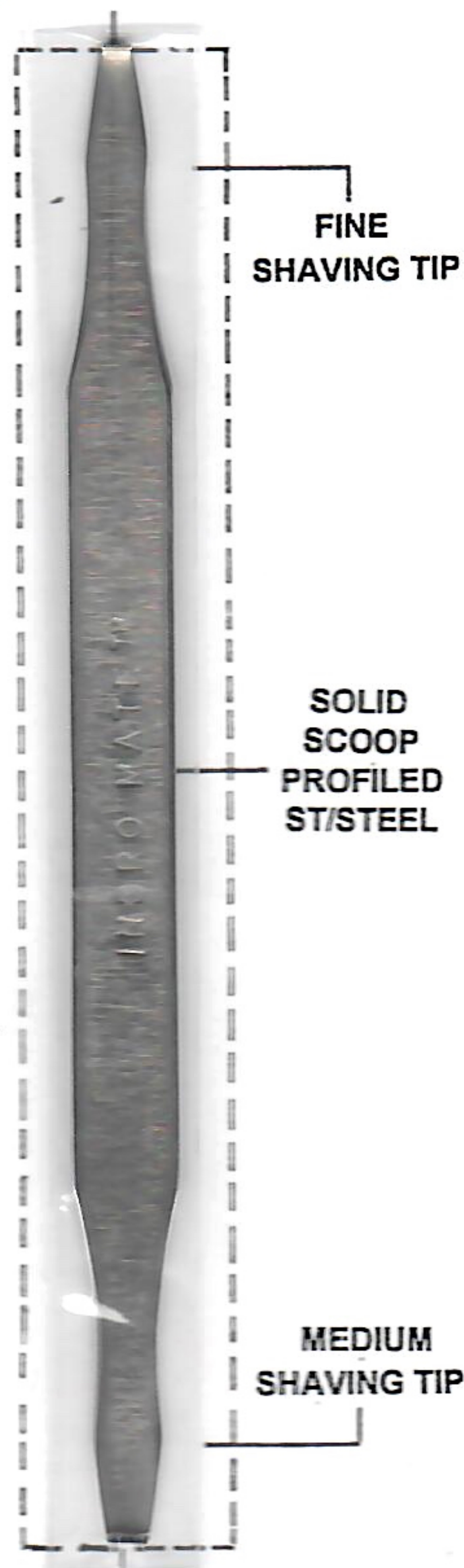

DIRECTIONS FOR USE

1. • CLEAN PROBLEM AREA THOROUGHLY
• HOLD BACK TISSUE AWAY FROM NAIL
• INSERT TOOL TIP BETWEEN NAIL LABIA AND NAIL EDGE WITH TIP FACING AGAINST NAIL
- 2&3. • STARTING FROM BASE OF NAIL, APPLY PRESSURE TO INGRO MATE AGAINST NAIL EDGE AND DRAW INSTRUMENT TOWARD YOU, SHAVING AWAY OVERGROWN NAIL EDGE
4. • CLEAN AREA THOROUGHLY AFTER TREATMENT USING STERILE ALCOHOL SWABS.
5. • MEDIUM BLADED TIP MAY ALSO BE USED TO HORIZONTALLY SHAVE AWAY NAIL SURFACE IRREGULARITIES OR PROVIDE THINNING OF NAIL PLATE.

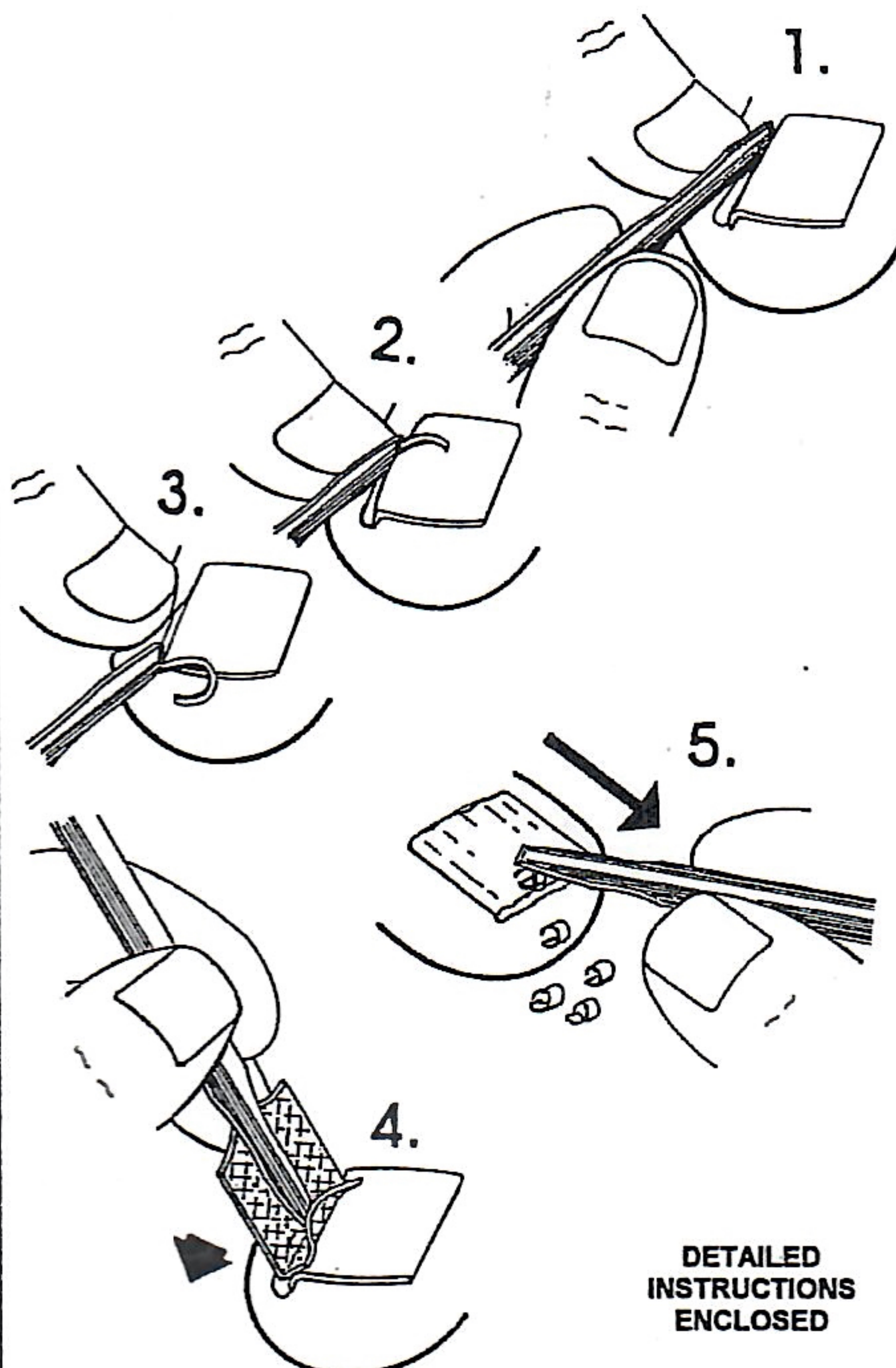

DETAILED
INSTRUCTIONS
ENCLOSED

Ingro Mate™
PRECISION INGROWN
NAIL INSTRUMENT

PRODUCT BRIEF

AN EXCEPTIONAL NEW AND VERSATILE INSTRUMENT FOR USE BY THE PRACTITIONER TO EFFECTIVELY TREAT AND CONTROL INGROWN NAILS

IN THE QUEST FOR THE MORE EFFECTIVE TREATMENT OF THE COMMON INGROWN NAIL CONDITION, CANONBURY HEALTHCARE PROUDLY OFFERS THE INGRO MATE - A PRECISION INSTRUMENT DEVELOPED SPECIFICALLY FOR USE BY THE PRACTITIONER.

INGRO MATE - AN ESSENTIAL INSTRUMENT IN THE SELECTION OF EQUIPMENT FOR YOUR MEDICAL PRACTICE :

- IS TIME / EXTREMELY COST EFFECTIVE AND EASY TO USE
- IS DESIGNED FOR MINIMAL DISCOMFORT TO THE PATIENT
- WILL ONLY TREAT AREAS OFFERING RESISTANCE
- WILL THEREFORE NOT INJURE OR PENETRATE SOFT TISSUE
- IS DURABLE, WELL BALANCED AND COMFORTABLE TO USE
- IS CONSTRUCTED OF THE FINEST QUALITY STAINLESS STEEL

CHARACTERISTICS OF THE INGROMATE

- INGRO MATE IS USED SIMILARLY TO A SCRAPER TO SHAVE MATERIAL FROM THE NAIL EDGE ENSURING NARROWING OF THE NAIL AND SEPARATION FROM THE SKIN.
- EACH INGRO MATE IS CONSTRUCTED OF SOLID STAINLESS STEEL, SCOOP PROFILED FOR GRIP, BALANCE AND CONTROL
- THE DUAL, NON-CLOG SHAVING TIPS OF THE INSTRUMENT ARE BOTH TAPERED AND HOLLOW FORMED, ALLOWING EASY INSERTION BETWEEN NAIL AND TISSUE.
- THE FINE SHAVING TIP IS DESIGNED SPECIFICALLY FOR DELICATE NAIL EDGE REDUCTION, WHILST THE MEDIUM TIP MAY BE USED FOR GENERAL NAIL EDGE REDUCTION, INCLUDING HORIZONTAL NAIL PLATE SHAVING IN THE INSTANCE OF SURFACE IRREGULARITIES, OR TO OBTAIN FUNGAL MATERIAL FOR LAB TESTING.

MAINTENANCE OF THE INGRO MATE


- AS WITH ANY PRECISION INSTRUMENT, HANDLE WITH CARE.
- DO NOT INJURE SHAVING TOOTH TIPS.
- STERILIZE PER AUTOCLAVE OR WITH CHEMICAL SOLUTIONS AS RECOMMENDED FOR MEDICAL INSTRUMENTS.

OTHER PRODUCTS AVAILABLE:

- THE PEDIPAL *Pedipal*
• ONE PIECE STAINLESS STEEL CONSTRUCTION INCORPORATING AN EPOXY RESIN HANDLE WITH A MEDIUM AND FINE SHAVING TIP
- THE INGROTOOL 1 OR 5 PIECE INSTRUMENT SET
• THESE EXCLUSIVE INSTRUMENTS ARE ALSO FOR THE EFFECTIVE TREATMENT OF INGROWN NAILS AND COMPRISE PRE-TREATED, KNURLED ALUMINIUM HANDLES HOUSING INTERCHANGEABLE COMBINATIONS OF THE FOLLOWING STAINLESS STEEL SHAVING TIPS : FINE, MEDIUM, BROAD AND A BACK SPATULA.
NOTE - THE ONE PIECE INGROTOOL HAS FIXED TIPS
• CANONBURY HEALTHCARE DISTRIBUTE THE ABOVE INSTRUMENTS


EVERY CARE HAS BEEN TAKEN IN THE DESIGN AND MANUFACTURE OF OUR INSTRUMENTS AND WE ARE CONFIDENT THAT YOU WILL BE ENTIRELY SATISFIED WITH YOUR PURCHASE.

PRODUCT SPECIFICATION



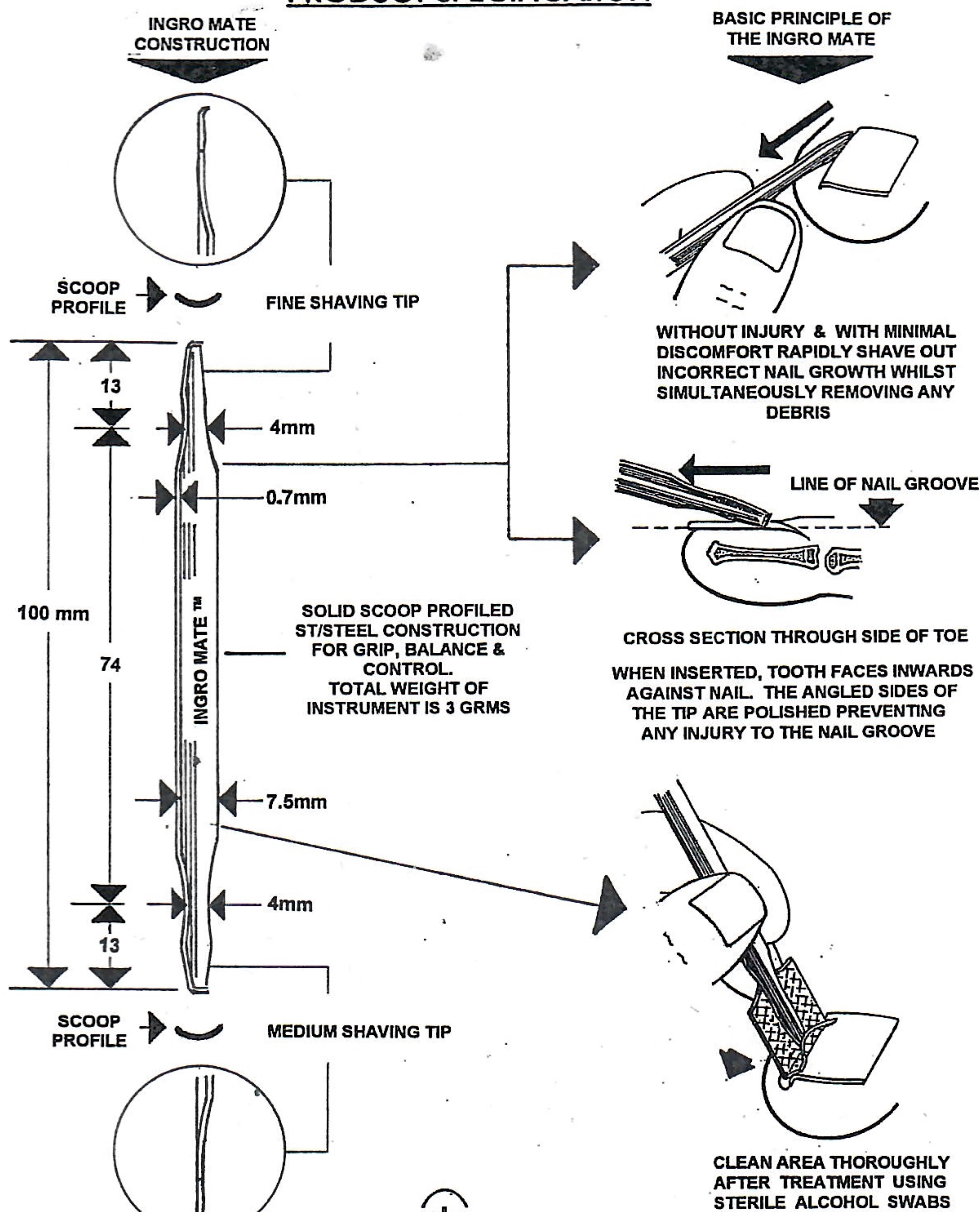


USAGE EXAMPLES

NAIL IS GROWING DOWN INTO TOE AND TISSUE RESULTING IN INFLAMMATION

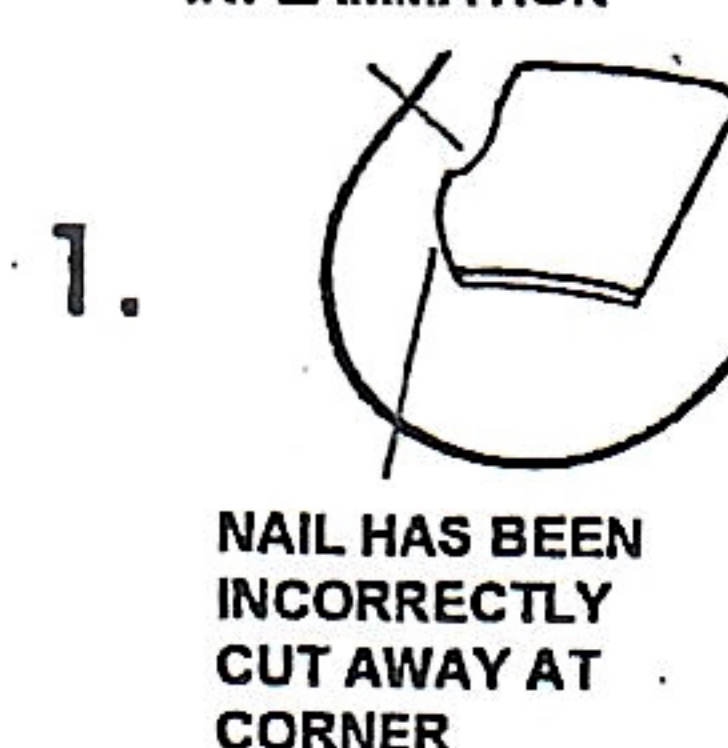

NAIL IS GROWING INTO SIDE OF TOE AND TISSUE RESULTING IN INFLAMMATION

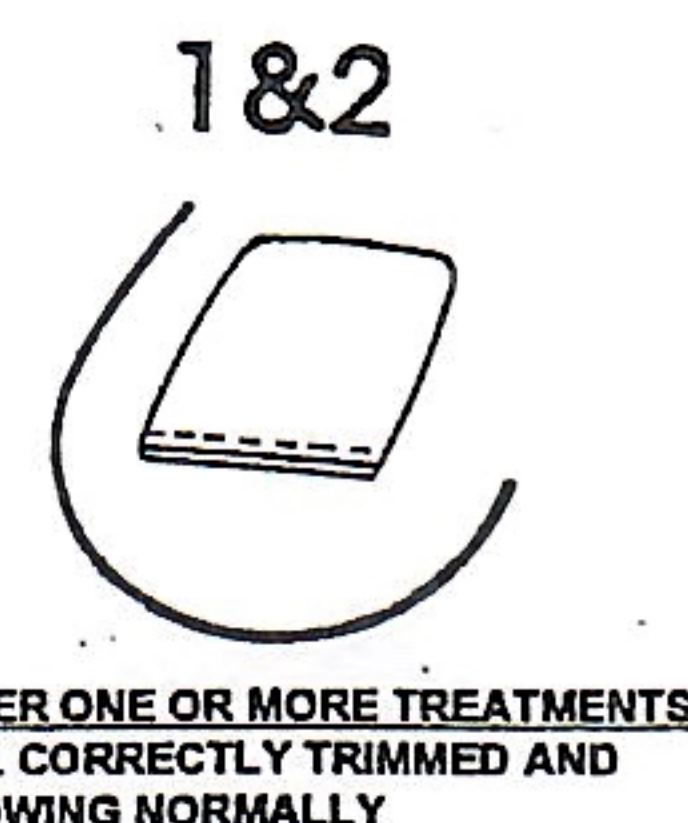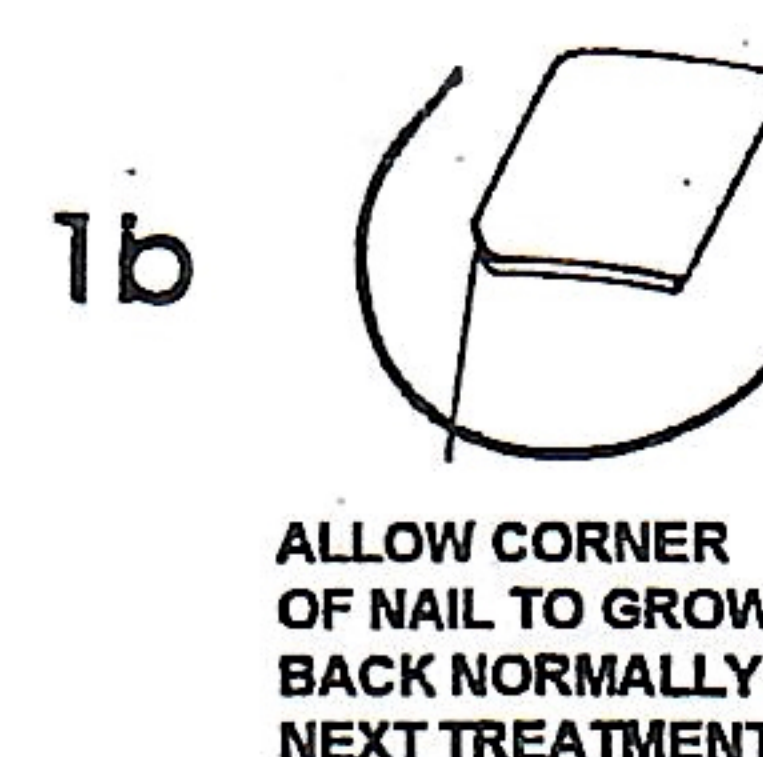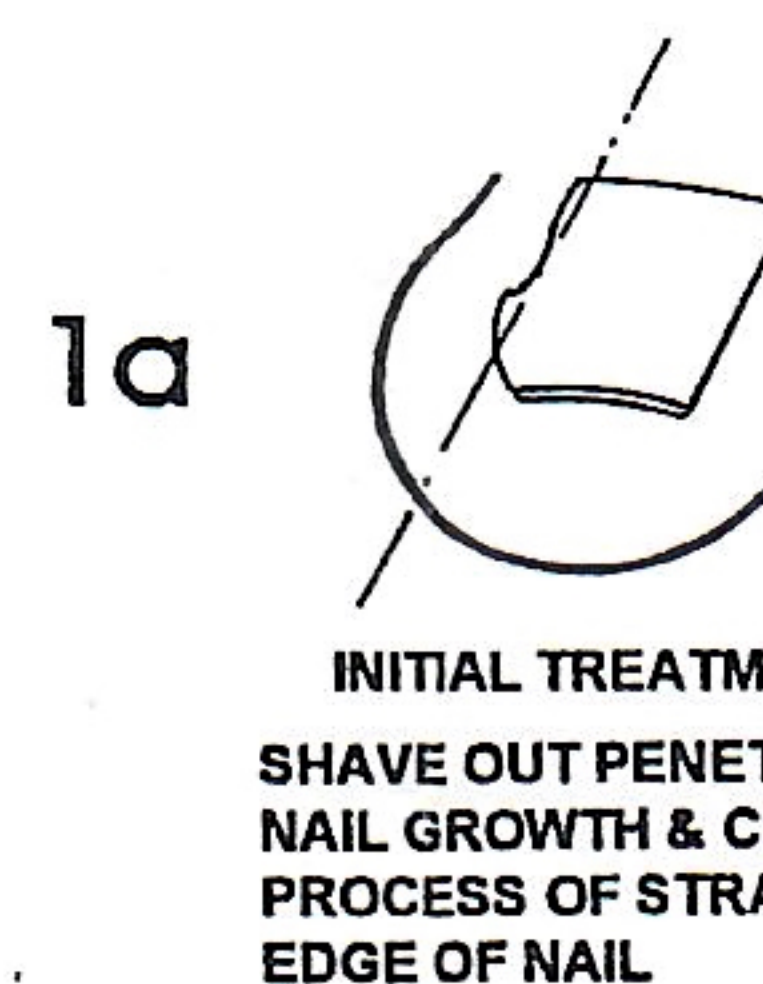

DISTRIBUTED BY :

canonbury
healthcare

RE-ORDER
DETAILS

2 ST JAMES RD, BRACKLEY
NORTHANS NN 13 7XY

PHONE 01280 706 661
FAX 01280 706 671
EMAIL canonbury@canonbury.com

DATE OF PUBLICATION OF INSTRUCTION LEAFLET : AUGUST 2006